



Implant Solutions



Exporting from
Acteon Viewer

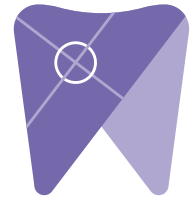


1. Open the CD or USB and click “start.”

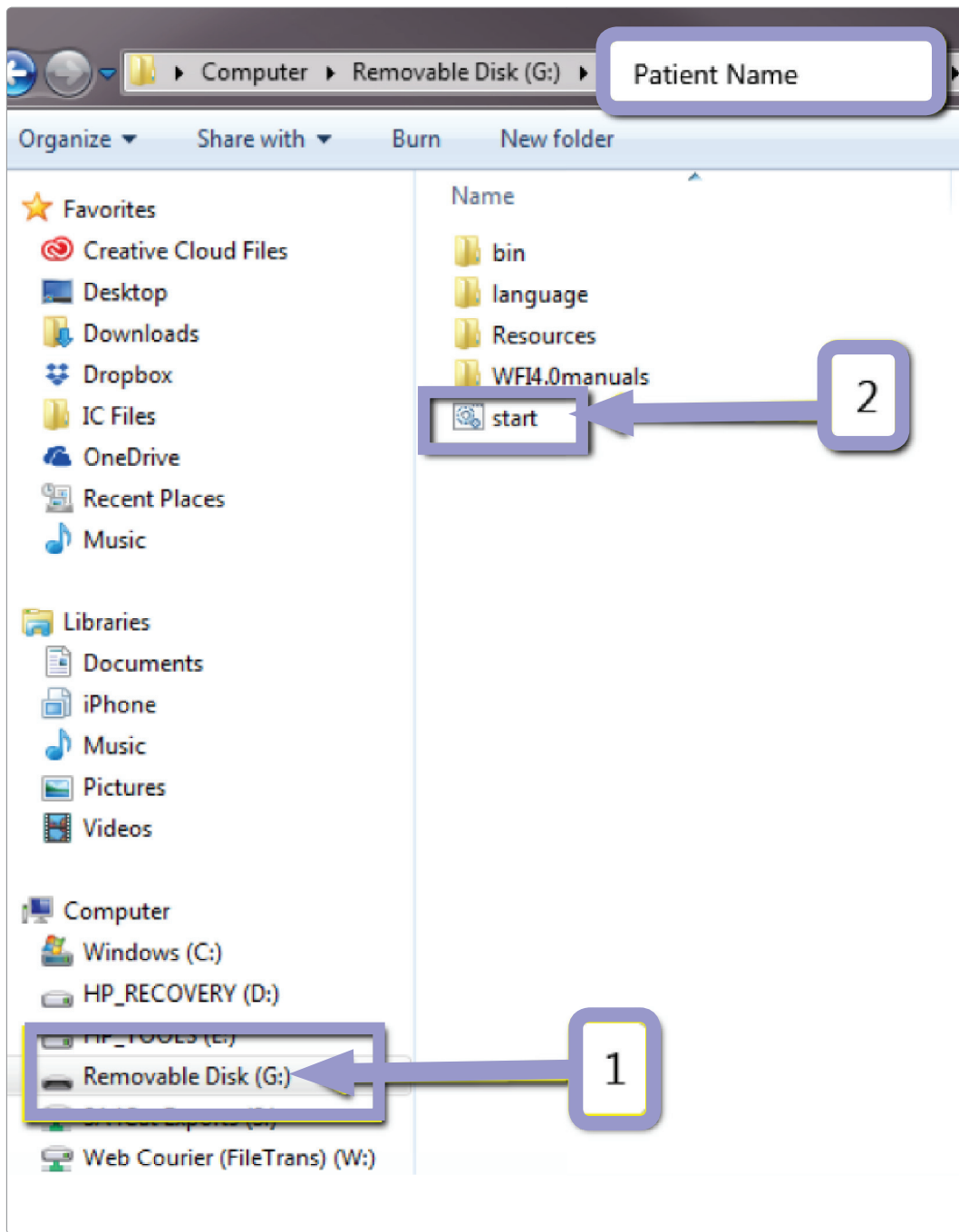


2. Once the CBCT loads in the AIS3DApp, click on the folder icon under “DICOM series.” Next, click on “zip DICOM series.”

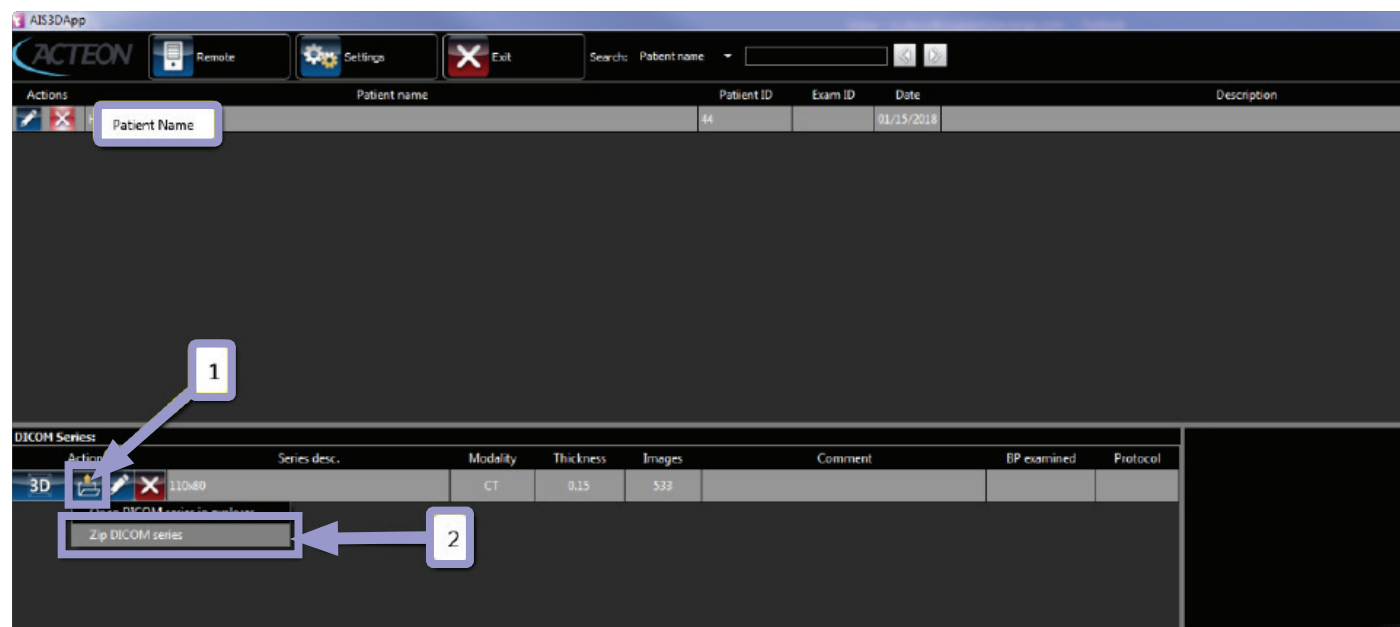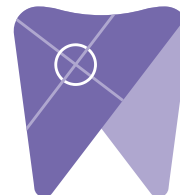


3. Save the file to your desktop and type the patients name into the “file name” field. Click “save.”

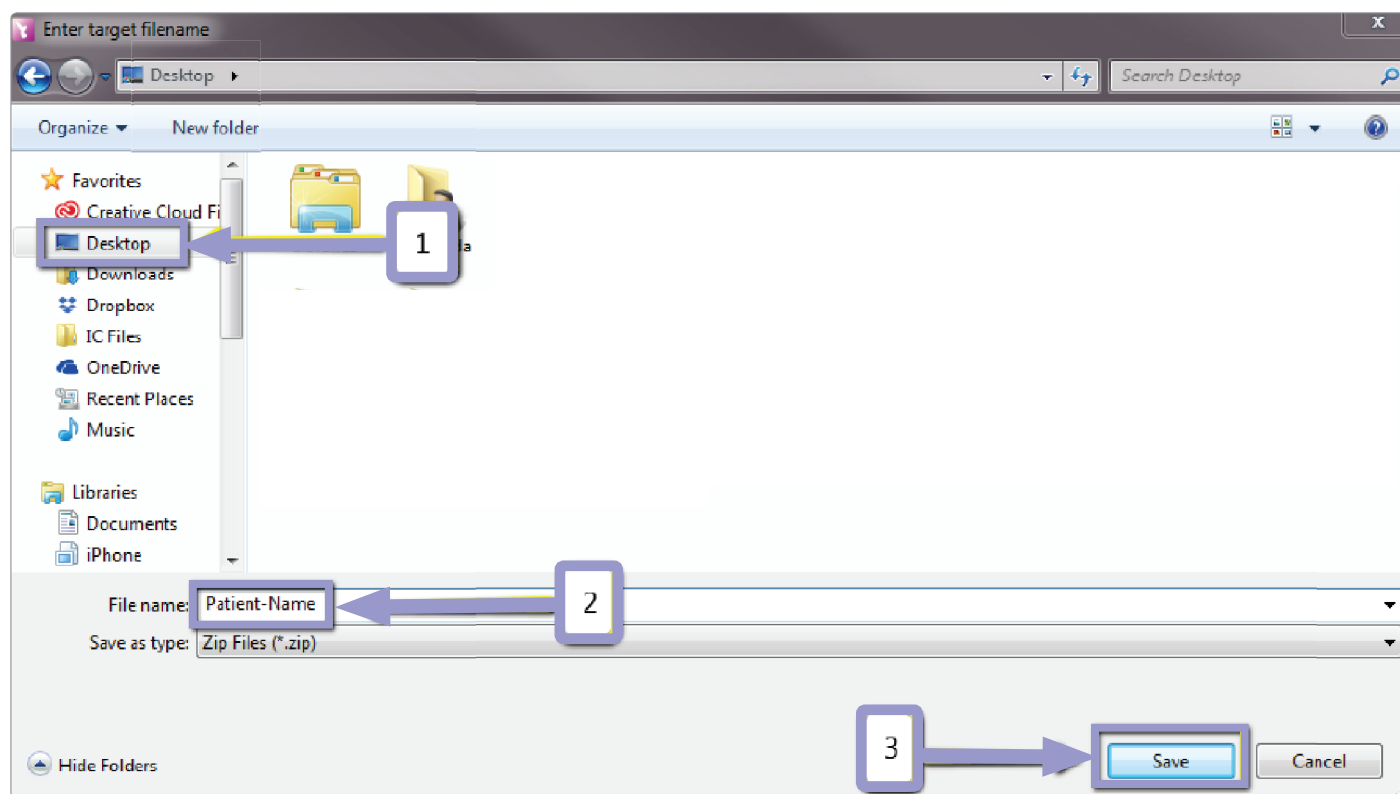

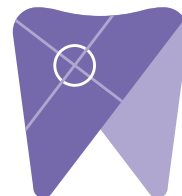

4. In the upper-right-hand corner, you will see a loading box. There will not be an alert when it has finished exporting.

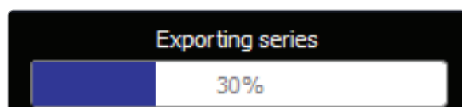

5. Visit Implant Solutions website at www.solutionsforimplants.com and select "Log in" in the top menu bar. Enter username and password and complete the following:

- 1) Select "new case" and enter patient name
- 2) Click on "add" next to "restorations" and select "Implant Solutions" from the group
- 3) Select a product from the down down menu
- 4) Select a tooth number and click "ok"
- 5) Click "add" next to "documents."

When prompted, select the zipped folder you created earlier.

Select the "I agree to the terms and conditions of sending this case to the lab" checkbox at the bottom of the page.

- 6) You will get a status update as the case uploads.

Note: If you do not have an account created, select "register" and complete the form. Implant Solutions will be notified of your registration. You will receive a confirmation email once the registration process has been completed. Contact Implant Solutions at 1.800.995.0626 if you have any questions.