


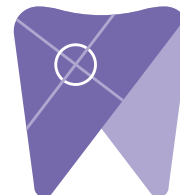


Implant Solutions


Exporting from
NewTom




1. Right click on your desktop and select “new” and click on “folder.”
Rename the folder to match the patient’s name.


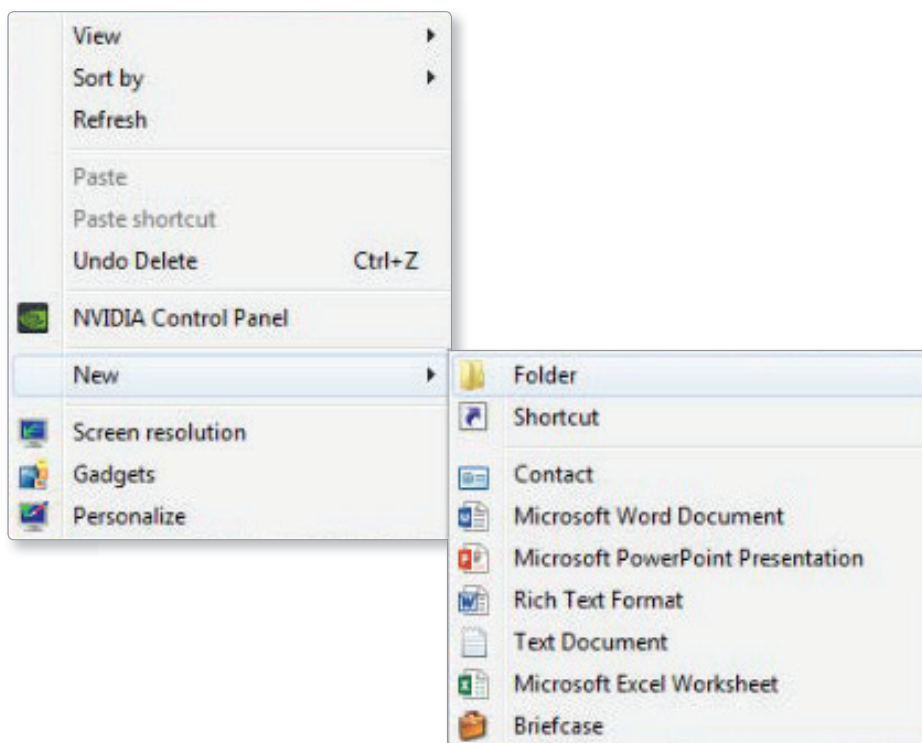



2. Open the patients scan. Next, select “file,” “save in DICOM format,” “sequence of axial images,” and select “free matrix.”

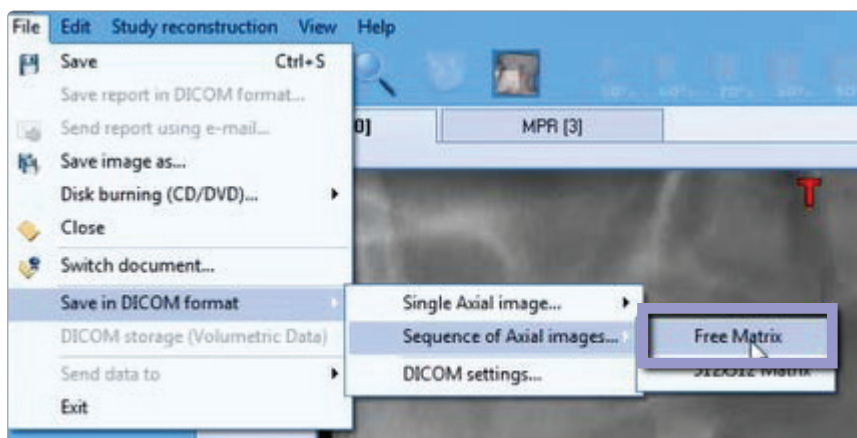

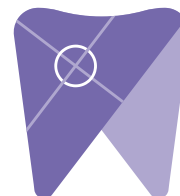



3. Select the folder you created on your desktop and click the “ok” button twice to export images.

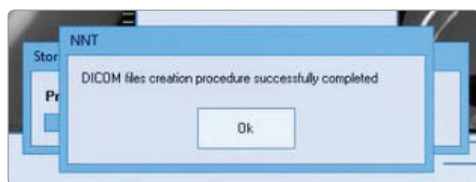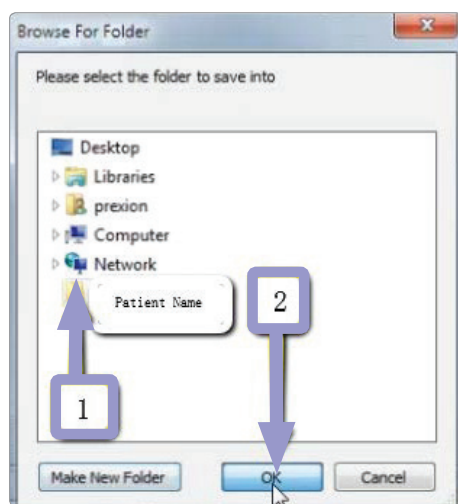

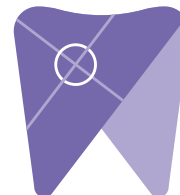


4. Open the patient's folder on the desktop and verify the .dcm files were exported correctly. There should be several hundred files.

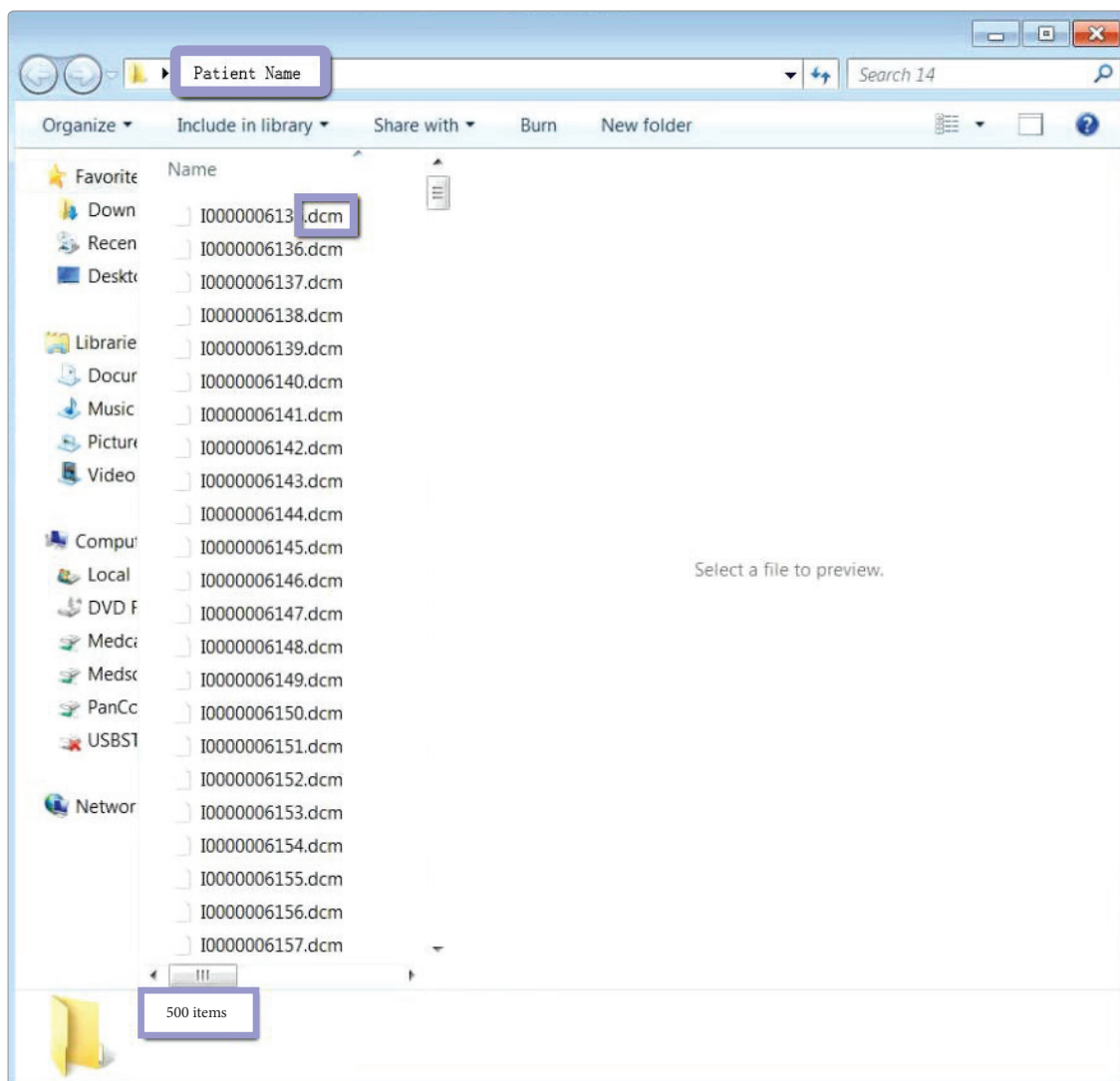

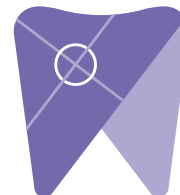

5. Right click on the patients folder and select “send to,” and “compressed (zipped) folder”. A duplicate folder will be made with either a zipper on it, or a blue “z.” (If you are following a Dual Scan Protocol, please zip scans individually and label accordingly; example “Patient Scan” and “Denture Scan”)

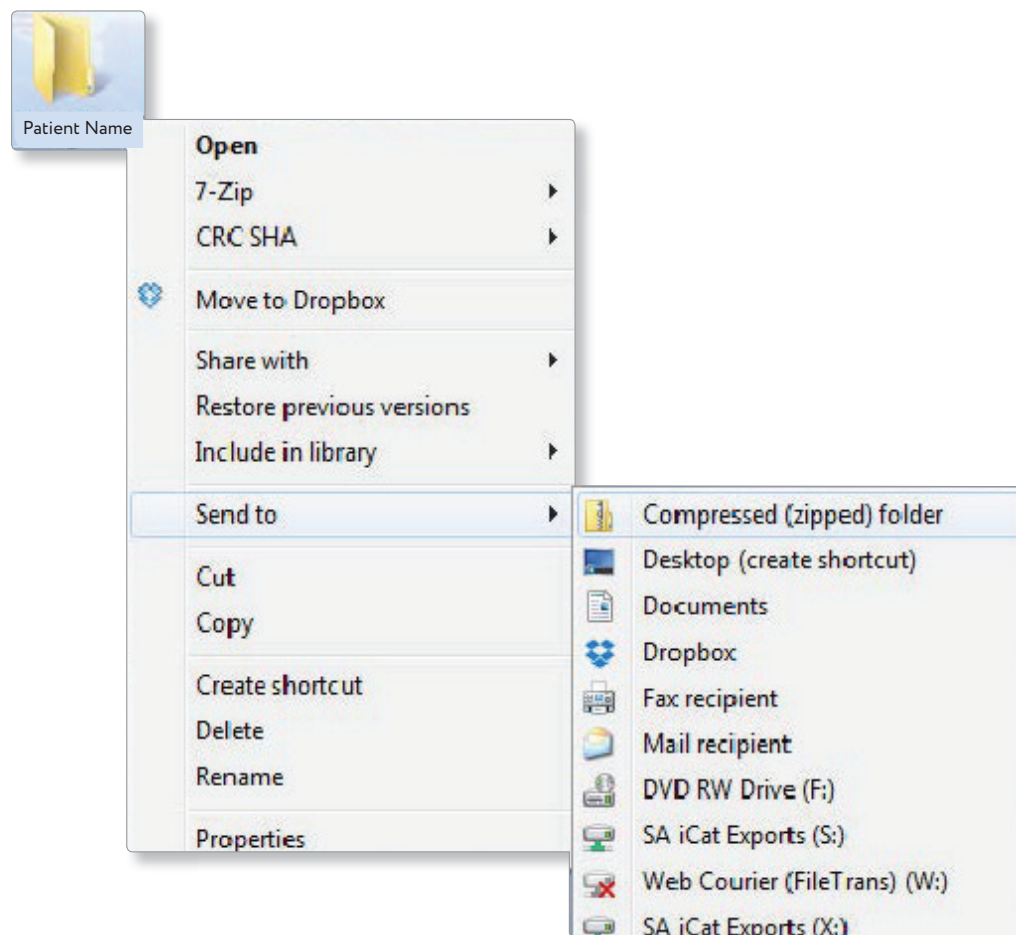

6. Visit Implant Solutions website at www.solutionsforimplants.com and select “Log in” in the top menu bar. Enter username and password and complete the following:
- 1) Select “new case” and enter patient name
 - 2) Click on “add” next to “restorations” and select “Implant Solutions” from the group
 - 3) Select a product from the down down menu
 - 4) Select a tooth number and click “ok”
 - 5) Click “add” next to “documents.” When prompted, select the zipped folder you created above. Select the “I agree to the terms and conditions of sending this case to the lab” checkbox at the bottom of the page.
 - 6) You will get a status update as the case uploads.

Note: If you do not have an account created, select “register” and complete the form. Implant Solutions will be notified of your registration. You will receive a confirmation email once the registration process has been completed. Contact Implant Solutions at 1.800.995.0626 if you have any questions.