



Implant Solutions

VaTech (Ez3D-i)

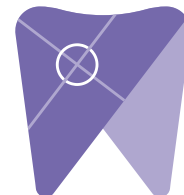






1. Right click on your desktop and select “new” and click on “folder.”
Rename the folder to match the patient’s name.

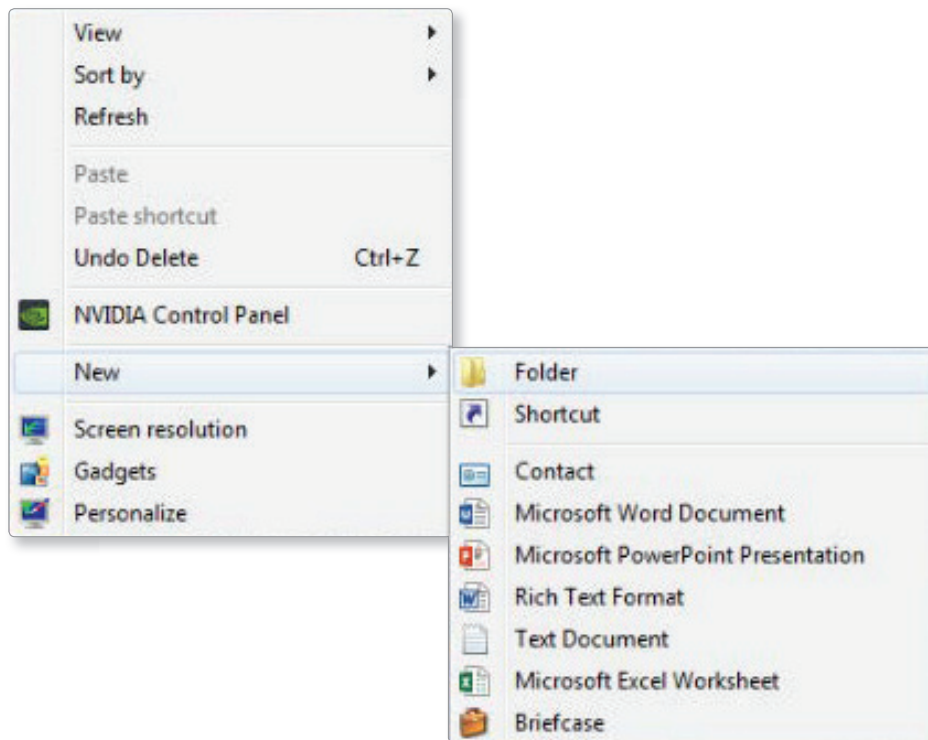


2. In your VaTech, open the patient scan. Click on “main menu” and then “export”.



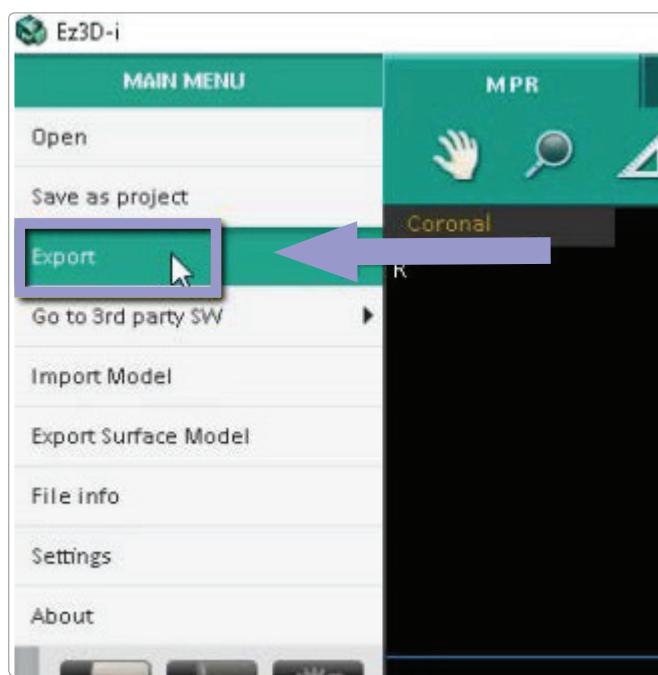


3. A dialogue box will appear. Select “local disk” for “save type” and “lossless” for compression. Then check the items that you would like to export. Implant Solutions requires the “DCM Files”.

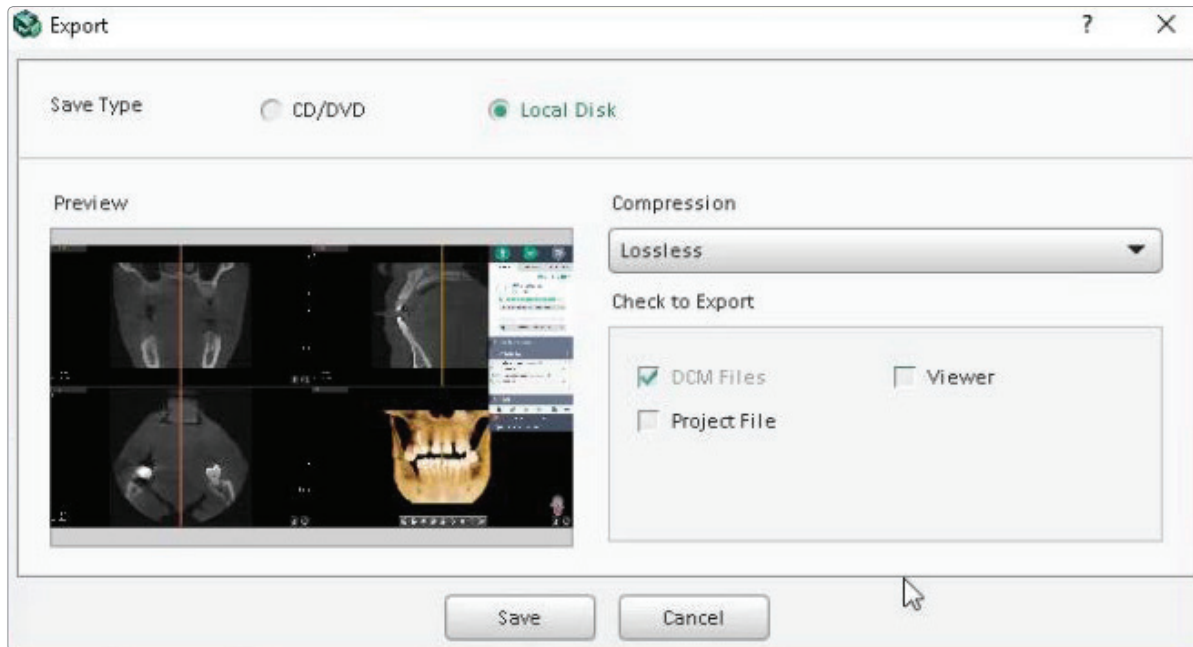
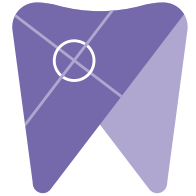


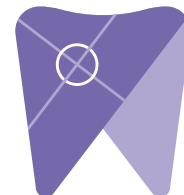



4. Select the folder you just created on the desktop, and click “select folder.”
The DICOM files will begin to export. Once the files are exported, you will get a notification. Next, click “ok.”

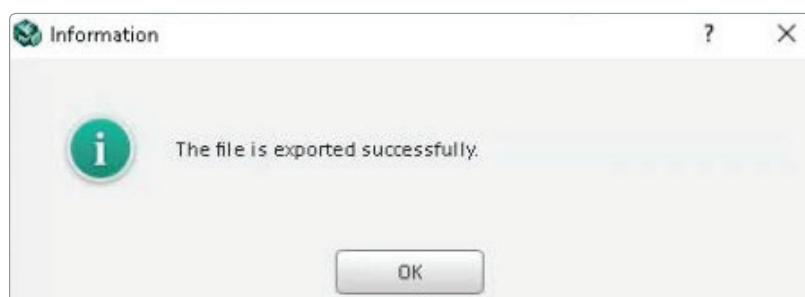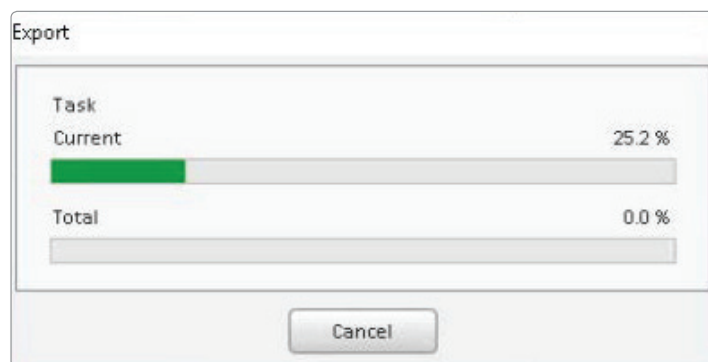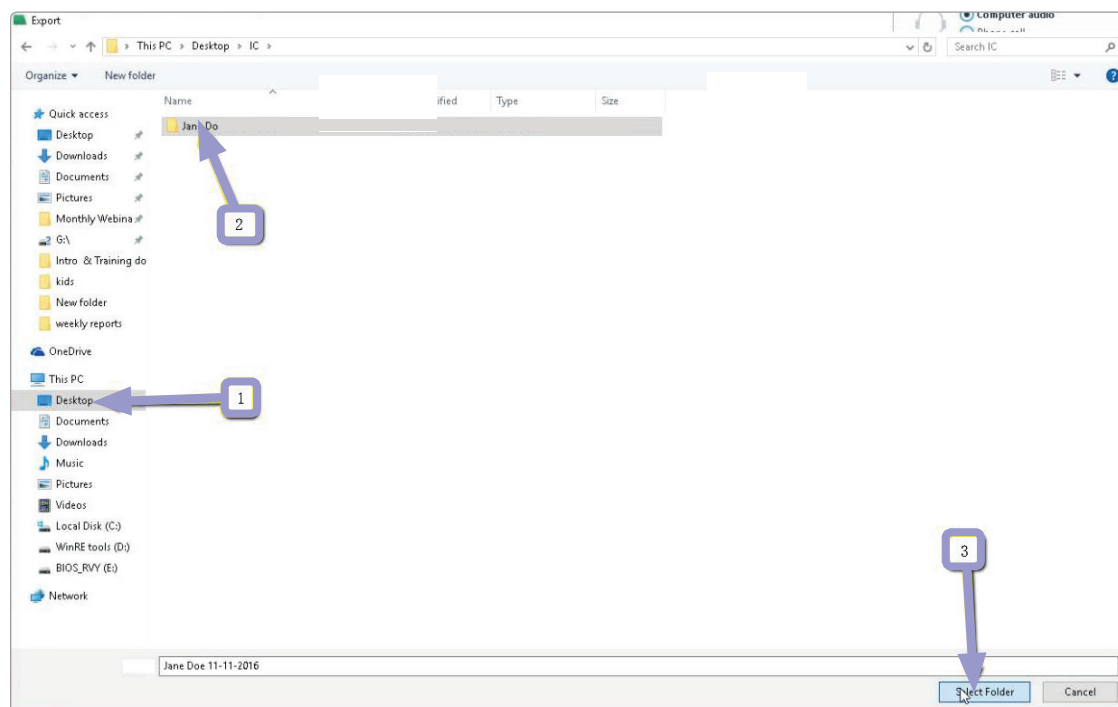

5. After the DICOM exports, go to the patient folder and verify the .dcm files were exported correctly. There should be several hundred files.

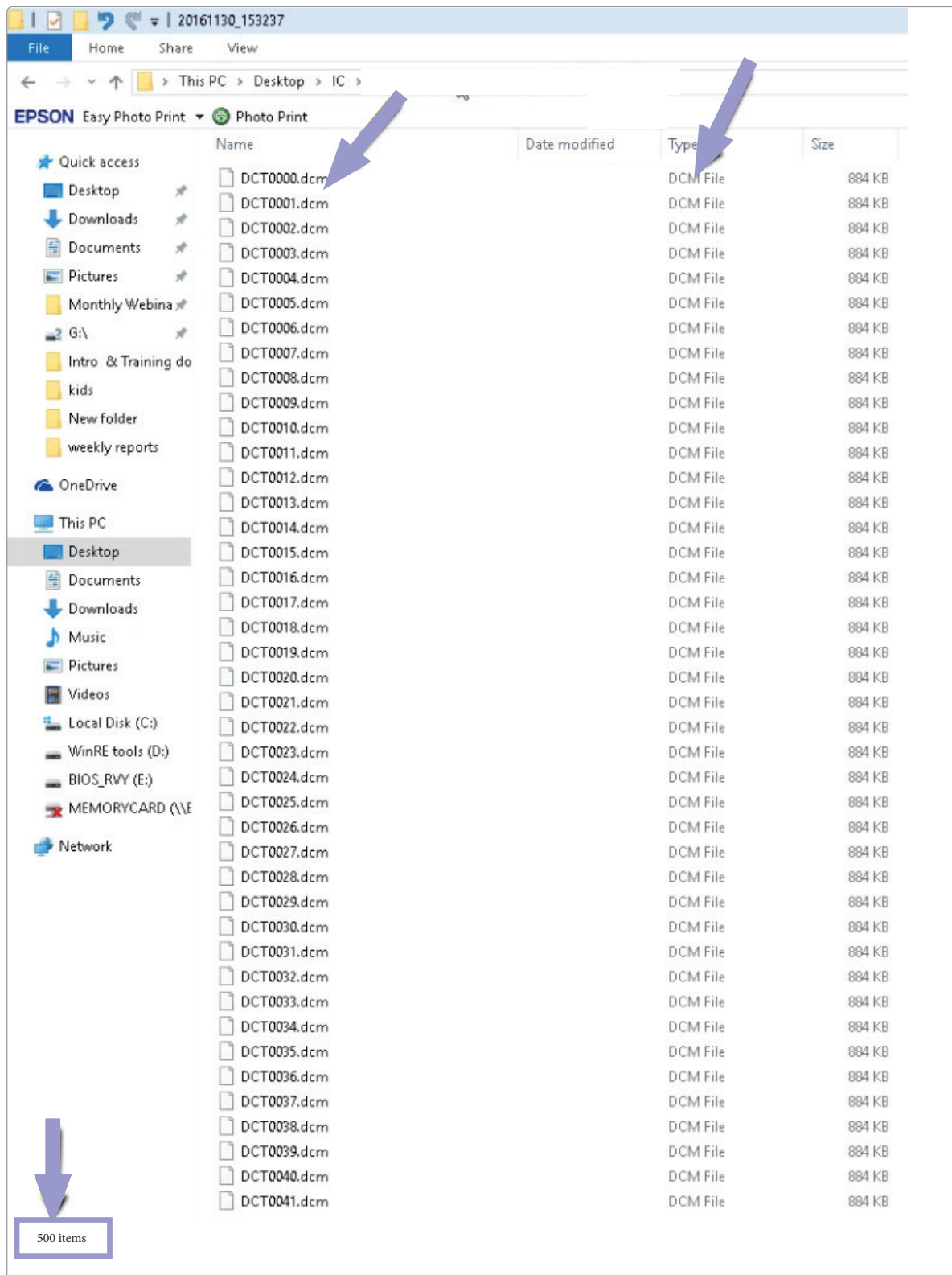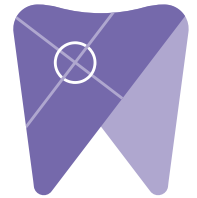

6. Right click on the patients folder and select “send to,” and “compressed (zipped) folder”. A duplicate folder will be made with either a zipper on it, or a blue “z.” (If you are following a Dual Scan Protocol, please zip scans individually and label accordingly; example “Patient Scan” and “Denture Scan”)

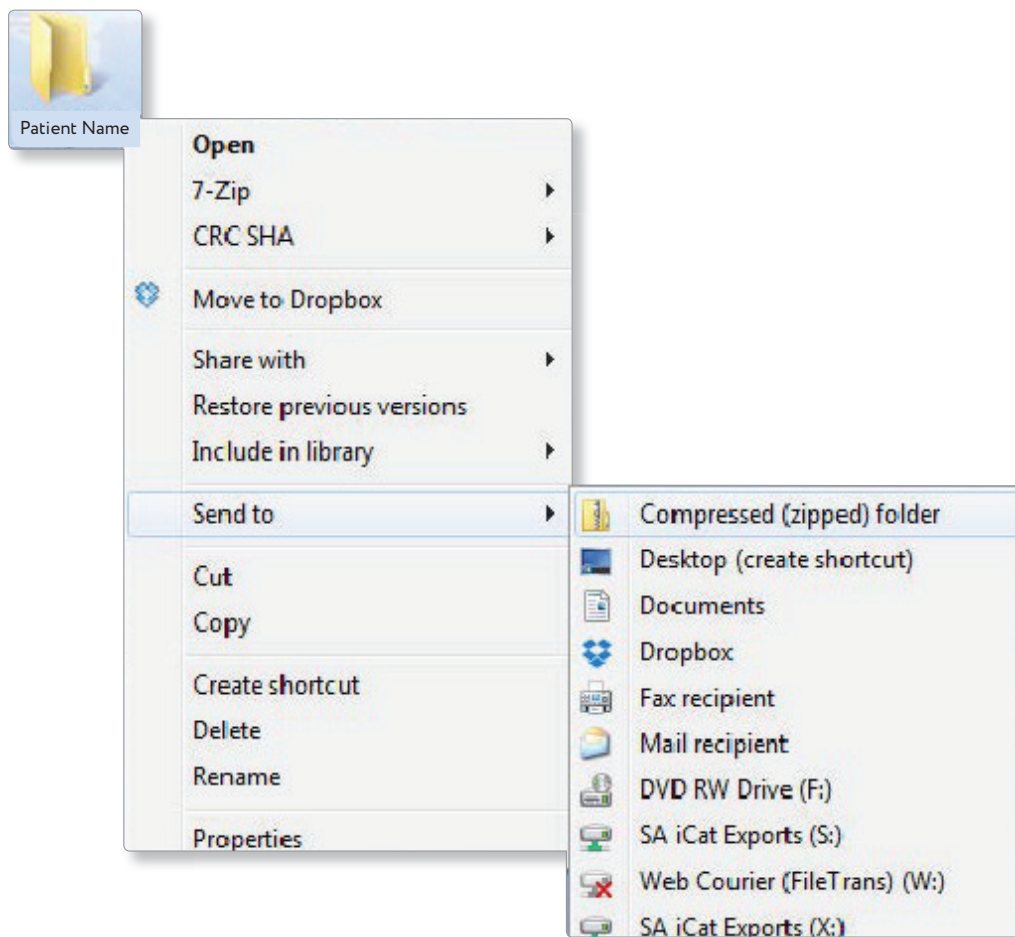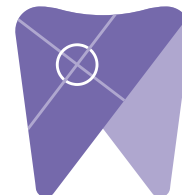

7. Visit Implant Solutions website at www.solutionsforimplants.com and select “Log in” in the top menu bar. Enter username and password and complete the following:

- 1) Select “new case” and enter patient name
- 2) Click on “add” next to “restorations” and select “Implant Solutions” from the group
- 3) Select a product from the down down menu
- 4) Select a tooth number and click “ok”
- 5) Click “add” next to “documents.” When prompted, select the zipped folder you created above. Select the “I agree to the terms and conditions of sending this case to the lab” checkbox at the bottom of the page.
- 6) You will get a status update as the case uploads.

Note: If you do not have an account created, select “register” and complete the form. Implant Solutions will be notified of your registration. You will receive a confirmation email once the registration process has been completed. Contact Implant Solutions at 1.800.995.0626 if you have any questions.